

Coil holder

User manual

General information

The equipment is specially developed for holding the coil allowing accurate positioning.

Thanks to its extended length, the coil holder allows the patient to be imaged in the same position as they will be in treatment.

The coil holder offers flexible height adjustment, conveniently marked with numerical positions for easy documentation on patient setup cards.

Storage and cleaning

Store in a clean, dry place away from direct sunlight.

Wipe thoroughly with a neutral antiseptic cleaner.


Allow to dry thoroughly before next use.

Product specifications

Subject	Value
Dimensions (L x B x H)	250x300x340mm
Weight	2,5 kg
Lifetime for the Board, based on the assumption that, service and maintenance has been carried out according to this manual.	20 years
Working temperature	$+5 < T_{amp} < +30 \text{ }^{\circ}\text{C}$
Environment	Only dry indoor conditions
Lightning over the board	Min. 200 Lux
Use in explosive environment (ATEX)	May not be used in ATEX areas
Use in food industry	May not be used in the food industry

Setting up the equipment

Attach and lock the coil holder to the board with these brackets, at the back side by pushing bracket pin into holes in the bracket.

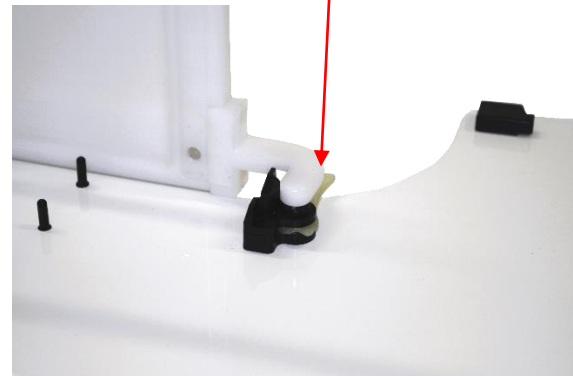

Adjusting the height

The height can be adjusted by moving the bracket to the left.

When the height has been adjusted, move the bracket to the right.

Now the bracket is locked.

